

CryoClear®

Repair. Renew. Rejoice.

FROM BEFORE AND AFTER
TO BEFORE AND **WOW.**



The all new, clearly better cryotherapy solution

CryoClear® uses Carbon Dioxide which freezes at -79°C to a depth of no more than 2mm into the skin, delivering a very safe and clinically effective treatment to superficial skin indications. CryoClear's high efficacy - over 95% - combined with less destruction of healthy tissue makes it perfect for treating age spots, sun spots, and skin tags. Whether used as a stand-alone treatment, or as a combination treatment to chemical peels, facials, micro-needling or dermaplaining, CryoClear offers a better, faster cryotherapy solution with significant benefits for clients and estheticians.

SKIN TAGS | Treatment Time: One 5 second treatment (base of Stalk)



AGE SPOT | Treatment Time: One 5 second treatment



Because CryoClear superficially treats indications that live within the epidermis, healthy skin will remodel to natural skin color as melanin is produced and lifted to the upper layers of the epidermis. This may take weeks to months following treatment.

CryoClearRx.com

CryoConcepts, LP • 205 Webster Street, Bethlehem, PA 18015 USA • 855.355.CRYO (2796)
CryoClear® name and logo are registered trademarks of CryoConcepts, LP. U.S. Patent #8,647,337. International patents pending.

QUICK-START GUIDE

Activate Pen

STEP 1




Remove CryoClear® from sealed pouch.



STEP 2



While grasping the pen with both hands, tighten the activation knob **CLOCKWISE** no more than one rotation. The blue dot on the activation knob should align with the arrow on the body of the pen.

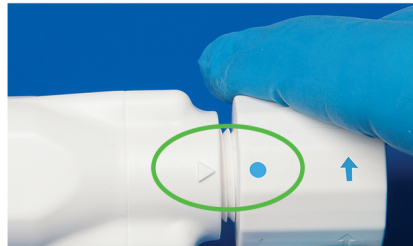

STEP 3



CryoClear® is now activated and ready to use.



DO NOT rotate activation knob in counter-clockwise direction once CryoClear® is activated.

QUESTIONS? CALL OUR SKIN CARE ADVISORS AT 1-855.355.CRYO (2796)

CryoClear®
Repair. Renew. Rejoice.

CryoConcepts.com

CryoConcepts, LP • 205 Webster Street, Bethlehem, PA 18015 USA • 855.355.CRYO (2796)

CryoClear® name and logo are registered trademarks of CryoConcepts, LP. U.S. Patent #8,647,337. International patents pending.

STEP 1



Position the client such that the surface to be treated is exposed and facing upwards.



PRE-TREATMENT HELPFUL HINT:

1. Be sure target site is well hydrated. Apply hyaluronic acid moisturizer (HA) to target site.


2. Roll a cooling/ice globe onto the target site for 20-30 seconds after HA is applied.

3. Move on to Step 2 above.

STEP 2



While holding CryoClear® in an upright position, hold the spray nozzle over the lesion at 2mm distance. Press the actuation button with the tip of the thumb to administer treatment.

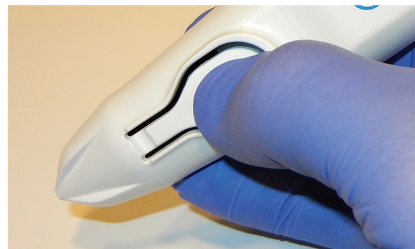

STEP 3



Spray CryoClear in a consistent slow motion while ensuring to treat the perimeter. White frost (as shown below) should be visible on the client's skin for entire duration of treatment.



POST-TREATMENT GUIDELINES

• Please refer to the Post Treatment Aftercare document and Advanced Treatment Protocols document for post-treatment recommendations

QUESTIONS? CALL OUR SKIN CARE ADVISORS AT 1-855.355.CRYO (2796)