

ADVANCED TREATMENT PROTOCOL

Although CryoClear® can be used alone, these protocols are suggested as ways to achieve optimal client experience.

PRE-TREATMENT SUGGESTED GUIDELINES



- 1 Apply hyaluronic acid moisturizer to target site
Benefit: Hydrates epidermis



- 2 Roll cooling/ice globe over target site for 20 - 30 seconds
Benefit: Cools target site

POST-TREATMENT SUGGESTED GUIDELINES

IMMEDIATELY POST-TREATMENT

- 1 Apply hyaluronic acid moisturizer
Benefit: Keeps epidermis hydrated
- 2 Roll cooling/ice globe over treated site for 20 - 30 seconds
Benefit: Reduces inflammation
- 3 Apply mineral sunscreen SPF30+
Benefit: Protects from sun

DAY 1 THROUGH DAY 14 – 21

- | | |
|--|----------|
| 1 Use gentle cleanser and cool water | PM |
| 2 Apply hyaluronic acid moisturizer | AM
PM |
| 3 Apply multi-level antioxidant serum | AM
PM |
| 4 Apply lipid-rich moisturizer
<i>NO retinol, glycolic or astringent ingredients!</i> | AM
PM |
| 5 Apply mineral sunscreen SPF30+ | AM |

AFTER 21 DAYS

- 1 After 21 days or when little to no pink remains, return to regular skin care regimen with glycolic, salicylic, retinol, etc.
- 2 Clients with tendency to hyper-pigment or who have sun damaged/aging skin - add tyrosinase inhibitor serum (bearberry extract, kojic acid or esterized Vitamin C aka C-Ester) with non-aggressive ingredients.
Note: Do not use tyrosinase inhibitor too soon. Wait at least 3 weeks before applying.

COMBINATION TREATMENT TIMELINE

TREATMENT	WAIT TIME
Facial (excluding any chemical peel >5% or mechanical exfoliation)	No wait time. Use CryoClear before serum is applied
Hydra / Microderm Infusion	Wait at least 5 days before treating with CryoClear
Chemical Peel	Wait at least 10 days before treating with CryoClear
Microdermabrasion	
Microneedling	
Dermaplaning	

Note: Any treatment or procedure that involves a chemical peel >5% or mechanical exfoliation should not be paired with CryoClear during the same client visit.

QUESTIONS? CALL OUR SKIN CARE ADVISORS AT 1.855.355.CRYO (2796)

CryoClear®
Repair. Renew. Rejoice.

CryoConcepts.com

CryoConcepts, LP • 205 Webster Street, Bethlehem, PA 18015 USA • 855.355.CRYO (2796)

CryoClear® name and logo are registered trademarks of CryoConcepts, LP. U.S. Patent #8,647,337. International patents pending.