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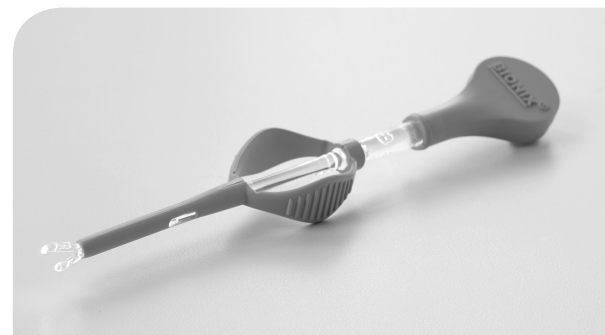
**REORDER NUMBERS:**

**#2750 - BOX OF 10 FORCEPS, 1 LIGHT SOURCE, 1 MAGNIFICATION LENS**

**#2775 - BOX OF 10 REPLACEMENT FORCEPS**

## LIGHTED FORCEPS FOR FOREIGN BODY REMOVAL - USER INSTRUCTIONS

Assembly/Disassembly:

1. To assemble the Lighted Forceps for Foreign Body Removal, insert the end of the forceps into the light source by aligning the pegs on the end of the forceps with the grooves inside the light source. Push down and twist FIRMLY clockwise until rotation stops. This will lock the forceps into place and activate the light.
2. To attach the magnification lens, snap the lens onto the neck of the light source so that the top of the lens angles towards the forceps tip.
3. To disassemble, push down on the forceps and rotate counter clockwise to remove the forceps from the light source. The light will go off when the forceps are removed.

Instructions for Foreign Body Removal

1. Examine the ear or nose to determine the location of the foreign body.
2. Assemble the Lighted Forceps for Foreign Body Removal according to instructions.
3. Insert the Lighted Forceps tip until it reaches the foreign body. Once there, squeeze the spring grip to grasp the object and gently pull from the orifice.
4. Repeat the exam and determine location of any remaining part of the foreign body.
5. Repeat the procedure until foreign body is removed.
6. Remove and dispose of the single-use forceps.

**CAUTION:** Discontinue curettage immediately if bleeding, irritation, or other trauma to the ear canal or tympanic membrane occurs.

**CAUTION:** This product is a spring-loaded device; improper assembly may cause ejection of the ear curette.

**CAUTION:** DO NOT hold device by light source as accidental ejection of curette may occur if not assembled properly.

The Lighted Forceps for Foreign Body Removal tip is intended for single-patient use. The light source and magnification lens are designed for multi-procedure use. Do not dispose of the light source or magnification lens. A light source and lens are provided in each box of product.

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©2019 Bionix Development Corp. | 5154 Enterprise Blvd., Toledo, Ohio 43612